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THANK YOU SO MUCH FOR CHOOSING AMMO

Before you start using your WAVE for the first time, we recommend that you read these recommendations for use so that you can familiarize yourself with it. They are designed to help you from the beginning. Wave is the perfect ally for your intimate health. Elegant, soft and discreet, it has an ideal design for any woman's profile. It is made of medical grade silicone (in one piece, safe, soft touch, easy to clean and odorless), it is fully waterproof and has a sound intensity of less than 50dB.

WHAT IS WAVE

Wave is a device for stimulating the clitoris through sonic waves instead of the classic vibrations. These waves bounce off in the device itself and reach your body to stimulate it without the need for contact.

BENEFITS OF USING WAVE:

- Recommended for women who do not reach or have trouble reaching orgasm.
- It helps self-knowledge at the anatomical level and improves sexuality, not focusing on penetration as the only way to have sex.
- Stimulation without direct contact with the clitoris. This makes it less likely to cause hypersensitivity and irritation than vibrating stimulators.
- Perfect to discover new sensations.
- Improves sexual desire and motivates to resume erotic life, alone or as a couple.
- Facilitates relaxation. Orgasm is beneficial for the care and strengthening of the pelvic floor.
- It favors blood circulation in the pelvic area, and therefore improves natural vaginal lubrication, avoiding dryness.
- Improves the elasticity of the vaginal walls.
- Helps to achieve female orgasm and multi-orgasm.
- Provides feelings of pleasure and well-being.

PROGRAMS:

Wave offers 9 different modes of sonic stimulation that are easily changed using the ON / OFF button (red illumination). In 1-2-3-4 modes you can select 3 different intensity levels (Low, Medium and High) using the + or - buttons. (blue lighting).

Modes 1 Continuous stimulation patterns, as well as orgasm, have a useful analgesic effect in treating pain and hypersensitivity processes.

Modes 2, 3, and 4 Discontinuous stimulation patterns activate the erectile musculature of the clitoris and are therefore useful for treating atrophies. These tissues of the clitoris surround the posterior aspect of the urethra, supporting it and creating the urethrovaginal complex.

Modes 5 and 6 Continuously increasing and decreasing patterns promote increased blood supply to the vulva, including the entire clitoris and vagina. This is especially helpful in treating vaginal atrophy and other lubrication difficulties. By offering a variation in stimulation they are highly enjoyable.

Modes 7 and 8 Ascending stimulation patterns increase sensitivity and are therefore useful for treating lack of sensitivity, hypertone, difficulties reaching orgasm.

Mode 9 Random patterns produce highly stimulating, new and unexpected sensations that facilitate orgasm. Sometimes our body can get used to it and need more and more intensity. This program prevents this habituation syndrome by varying both rhythms and intensity.

THERAPEUTIC APPLICATIONS:


- Accurately locate the clitoris so it helps to know all parts of the vulva.
- It increases the connection with the pelvic area and facilitates proprioception, to be aware of the body's own response to stimulation.
- It increases the vascularity of the clitoris in all its extension and the vagina.
- It promotes natural lubrication, making it ideal for women with vaginal atrophy, cancer processes or difficult postpartum.
- It is an ally in the treatment of pain in sexual difficulties such as vulvodynia and dyspareunia.
- It quickly activates the orgasm reflex, which is why it is useful in treating difficulties in reaching orgasm.
- Very useful in the treatment of hypertone of the pelvis, atrophies of the clitoris and the urethra.
- It helps to treat anorgasmia or difficulties in reaching orgasm, ruling out organic causes.
- The activation of blood circulation and orgasm help to improve the tone and strengthening of the pelvic floor.
- It promotes a gradual approach to erotic life for those women who have abandoned sexual activity for fear of feeling pain, since Wave stimulates without contact.
- Motivates the woman to get a good learning and an adequate fulfillment of the work recommended by the professional.

WHAT MAKES WAVE SPECIAL:

- Small size (79mm)
- Lightweight, only 92gr.
- Discreet design.
- Silent, less than 50db
- Ergonomic. It adapts to your anatomy.
- Made of extremely soft medical grade silicone.
- Easy to clean.
- Easy to handle. Illuminated keypad with option to raise and lower intensity in modes with this option (1,2,3,4)
- Waterproof.

- 17 different routines. (9 different modes and 3 intensities (modes 1 to 4)) to have experiences new with each one.
- Reliefs drawn on the sides to facilitate its grip and avoid slipping.

DISCOVER WAVE: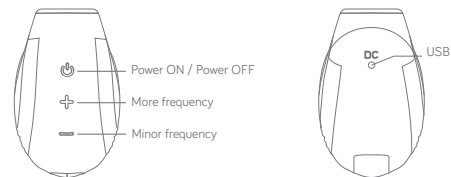


HOW IS IT USED

Before using it, charge your Wave. You will find a cable with a USB connection in the box. You must insert the cable into the charging port that you will find at the base of the Wave, it is indicated by the letters DC.

Spread your lips slightly to expose your clitoris. Position the Wave so that the mouthpiece is around the clitoris. Press the on / off button for 2 seconds (a red light will illuminate under this button). It is handled in a very intuitive way. Select the mode with the ON / OFF button and change intensity with the plus and minus buttons (a blue light will illuminate). Modes 1-2-3-4 are those that allow intensity change. Experience 100% pleasure. To turn off the device, press and hold the on / off button for at least 2 seconds.

Do not forget to clean the device well with warm water and mild soap before and after each use.

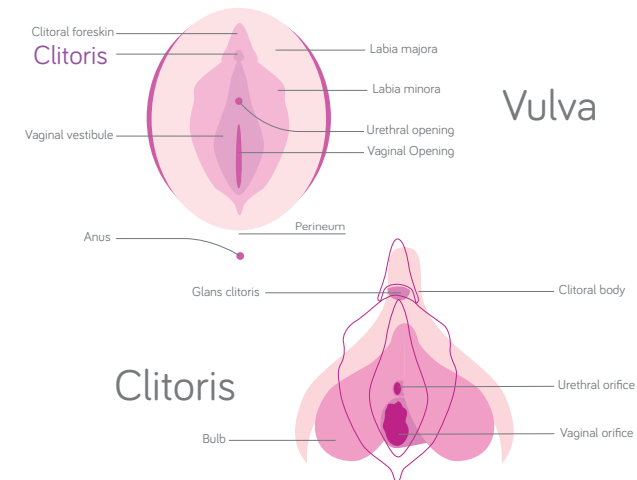Vulva

Clitoris

MODE 7 (fly low)

CONSTANT RISE FROM THE BOTTOM OF THE SUCTION INTENSITY FROM LOW TO HIGH WITHOUT INTERRUPTIONS 2 seconds up, 2 seconds up

MODE 8 (fly high)

CONSTANT RISE FROM THE BOTTOM OF THE SUCTION INTENSITY FROM LOW TO HIGH WITHOUT INTERRUPTIONS 2 seconds up, 2 seconds up


MODE 9 (rock& roll)

A RANDOM COMBINATION OF MODES 1 TO 8


MODE 1 (continuous)

CONTINUOUS SUCTION WITH THE POSSIBILITY OF VARYING INTENSITY WITH THE + AND - BUTTON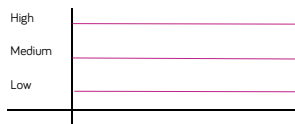


MODE 2 (short pulsations)

SUCTION BY PULSATIONS WITH POSSIBILITY OF VARYING INTENSITY WITH THE + AND - BUTTON 1 second on, 0,5 second rest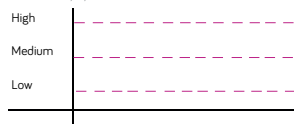


MODE 3 (medium pulsations)

SUCTION BY PULSATIONS WITH POSSIBILITY OF VARYING INTENSITY WITH THE + AND - BUTTON 2 seconds on, 0,5 seconds rest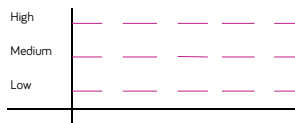


MODE 4 (long pulsations)

SUCTION BY PULSATIONS WITH POSSIBILITY OF VARYING INTENSITY WITH THE + AND - BUTTON 3 seconds on, 0,5 seconds rest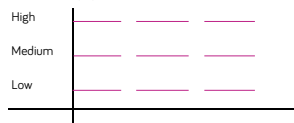


MODE 5 (long pulsations)

CONSTANT RISE AND FALL OF THE SUCTION INTENSITY FROM MEDIUM TO LOW WITHOUT BREAKS 2 seconds up, 2 seconds down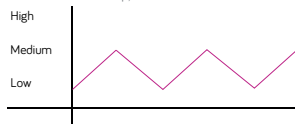

MODE 6 (high wave)

CONSTANT RISE AND FALL OF THE SUCTION INTENSITY FROM LOW TO HIGH AND FROM HIGH TO LOW WITHOUT BREAKS 2 seconds up, 2 seconds down

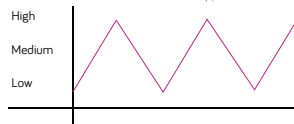CLEANING AND MAINTENANCE

Being made of medical grade silicone, Wave is totally safe for your body. It is made of a single piece of silicone to facilitate cleaning. We always recommend that before and after each use you wash it very well with warm water and mild soap for at least 30 seconds. You can dry it with a towel that does not leave residue or let it air dry.

If you use a lubricant, we recommend using a water-based intimate moisturizer (Aloe Vera-Chamomile Intimate Moisturizer does not stain, leaves no residue and is easy to clean). Do not use with silicone lubricant, oils or vaseline to avoid damaging the surface.

Keep your wave in a dry place. Do not expose it to extreme heat and do not leave it in the sun for a long time. Splashproof but not submersible, no water should enter the charging point. Store individually in its breathable bag made of organic cotton.

TECHNICAL CHARACTERISTICS

- Material: Medical Grade Silicone
- Measurements: 37mm x 59,5mm x 77mm
- Nozzle diameter: 14mm
- Weight: 92gr
- Flexibility: rigid
- Controller type: Button on / off, (red light) more intensity, less intensity (blue light)
- Silent: <50db.
- Splashproof
- 17 different routines. (9 different modes and 3 intensities (modes 1 to 4)) to have new experiences with each one.
- Power: lithium battery
- Input voltage: 5V
- Input current: 1-2A

If you want to know more, visit our website

www.ammolondon.com